**EN Dual-Spray MiniSpacer® Metered Dose Inhaler (MDI) Dispenser/Adapter**

- Clean, ready to use
- Latex free

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**NL Dual-Spray Minispacer® Dosisaërosol Dispenser/Adapter**

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1 = 1 Thayer MiniSpacer

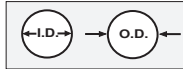
**THAYER MEDICAL**  
Thayer Medical Corporation  
Tucson, Arizona 85714-1961 U.S.A.  
www.thayermedical.com

**REF** 1024 MiniSpacer 22mm O.D./O.D.

**REF** 1025 MiniSpacer 22mm O.D./I.D.

**REF** 1071 MiniSpacer 15mm O.D./I.D.

**EC REP** MDSS GmbH  
Schiffgraben 41  
30175 Hannover, Germany 0482

**EN INDICATIONS FOR USE**

The Dual-Spray MiniSpacer is an adapter for administering metered dose inhaler (MDI) medication to ventilated patients. It may also be used with resuscitator bags, or inline with cool or heated humidification oxygen enriched T-pieces (blow-by) systems connected to tracheal/endotracheal tubes.

The Dual-Spray MiniSpacer is a single-use device not intended to be sterilized, reprocessed or reused.

**DIRECTIONS FOR USE**

1. Remove MiniSpacer from package.
2. Place the MiniSpacer in the desired location on the inspiratory side of the circuit.
3. Ensure a secure fit with all connections.
4. Cap the MDI port. Position the MDI port so that the MDI canister will be oriented vertically with the canister valve pointed downward
5. When ready for use, uncap the MiniSpacer. Shake the MDI canister and insert the canister into the MDI port. Do not discharge the MDI at this time.
6. At the beginning of inspiration, dispense the MDI by pressing the canister down. Deliver the dosage as ordered by the physician.
7. Upon completion of the treatment, remove the MDI canister and cap the MDI port.

**CAUTION**

Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare practitioner.

The Dual Spray MiniSpacer® MDI Dispenser/Adapter will not increment the counter on MDI canisters with integrated dose counters.

MINSPACER and THAYER MEDICAL LOGO are registered trademarks of Thayer Medical Corporation  
© 2009 Thayer Medical Corporation

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